



# ⚠ Pneumatic Stool Assembly Instructions – Without Back

1. Insert five Casters into stool Base.
2. Insert Pneumatic Gas Spring into Base.
3. Place stool Seat on Gas Spring. Sit on stool to secure Seat to Gas Spring.
4. Pull upward on Gas Spring Lift Handle to adjust height of stool.

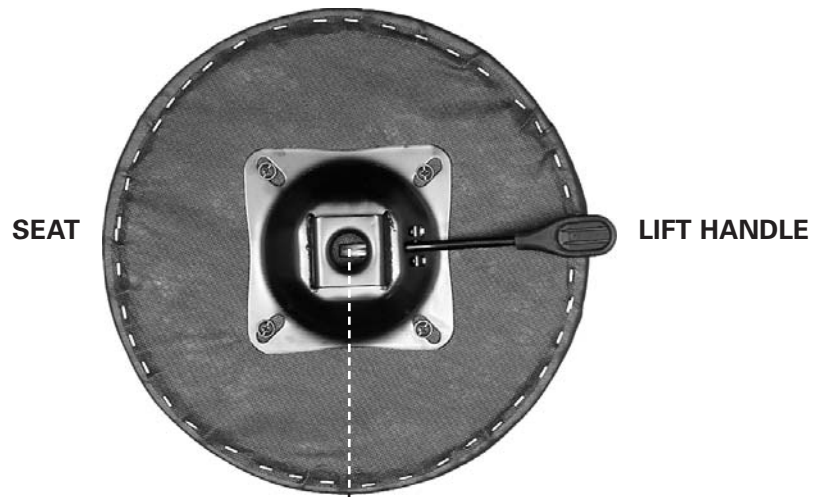

Qty	Description
1	Seat
1	Pneumatic Gas Spring
5	Casters
1	Base

PNEUMATIC GAS SPRING

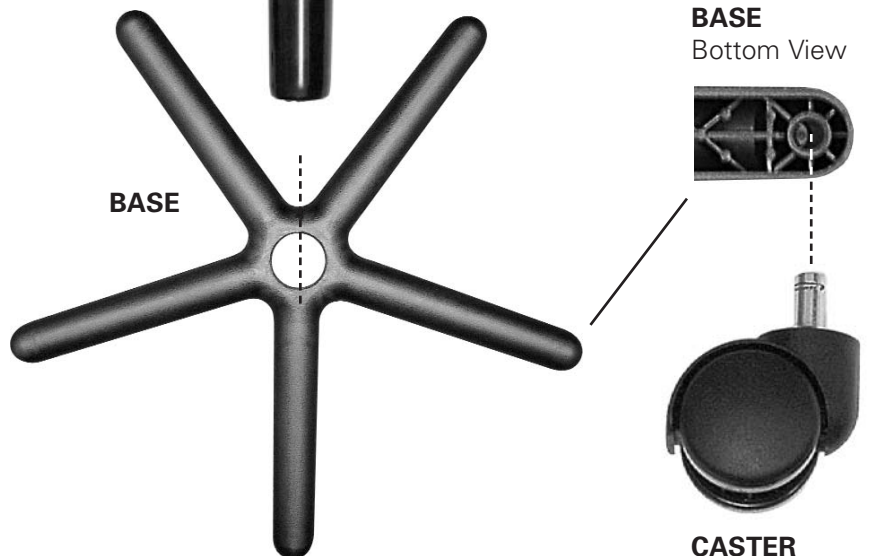

**CAUTION:** Do not stand on this product because serious injury could occur. Chattanooga Group, Inc. 4717 Adams Road, Hixson, TN 37343 USA is not responsible for injury or damage due to improper assembly and use of this product.



ISO 9001 CERTIFIED

4717 Adams Road • P.O. Box 489  
Hixson, TN 37343 U.S.A.

1-423-870-2281


1-800-592-7329 U.S.A.

1-800-361-6661 CANADA

www.chattgroup.com

# ⚠ Pneumatic Stool Assembly Instructions – With Back

1. Insert five Casters into stool Base.
2. Insert Pneumatic Gas Spring into Base.
3. Place stool Seat on Gas Spring. Sit on stool to secure Seat to Gas Spring.
4. Snap Back Support Bracket onto top end of Back Support Bar and secure to Back Support with two Screws. Tighten Screws with screwdriver (not included).
5. Attach Back Support assembly to Seat by inserting Back Support Bar into Back Support Bracket. Attach Washer onto Adjusting Knob and insert into Back Support Bar. Adjust Back Support position and tighten Adjusting Knob to secure.
6. Pull upward on Gas Spring Lift Handle to adjust height of stool.

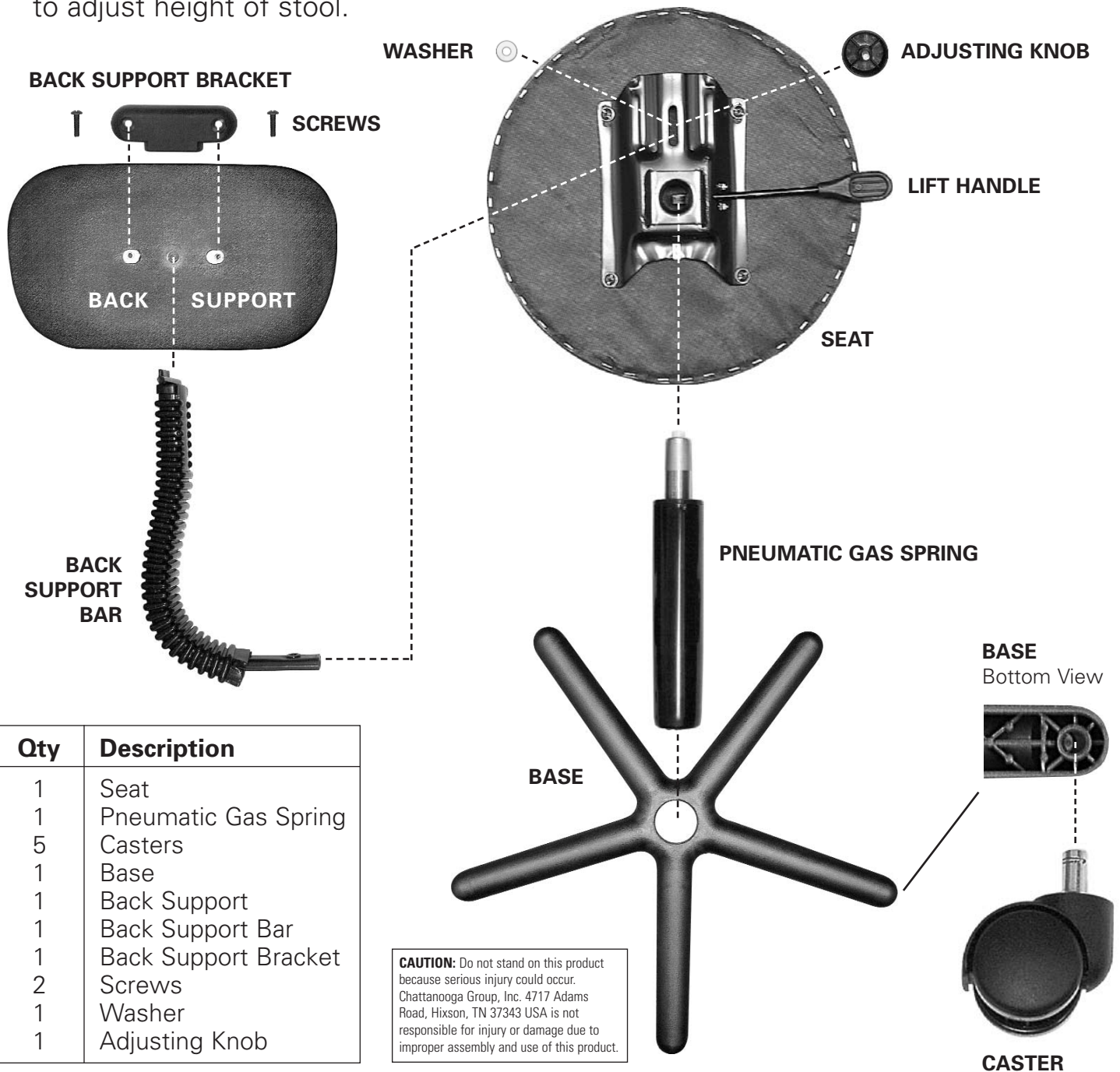

Qty	Description
1	Seat
1	Pneumatic Gas Spring
5	Casters
1	Base
1	Back Support
1	Back Support Bar
1	Back Support Bracket
2	Screws
1	Washer
1	Adjusting Knob

**CAUTION:** Do not stand on this product because serious injury could occur. Chattanooga Group, Inc. 4717 Adams Road, Hixson, TN 37343 USA is not responsible for injury or damage due to improper assembly and use of this product.